

INSTRUCTIONS AND USE

MAT[®]

Tourniquet



1 Place around limb, above injury.



2 "Click" buckle into place.



3 Pull strap down snugly.



4 Rotate turnkey to tighten and securely occlude blood flow.

SECURING MAT[®] - Option A



Wind strap around turnkey stem and fold turnkey down.

SECURING MAT[®] - Option B



Tuck strap underneath and tighten.

SECURING MAT[®] - Option C



Wrap excess strap over turnkey and tuck under.

Press the Release tab or lift on the buckle to release.

RELEASING MAT[®]



IMPORTANT

Make sure that **MAT[®]** is **FULLY RESET** before each application.

TO RESET

While pushing in **RED RELEASE** button, pull strap all the way out.